

Fitting Instructions

Model number: TWINTY1/2: FFP2



1



2



3



4



5

1. Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hands
2. Position the respirator under your chin with the nosepiece on the bridge of your nose placing the top strap high on the back of your head and the bottom strap below your ears
3. Using both hands, mould the nosepiece to the shape of the nose for a tight fit
4. Test the fit by cupping both hands on the respirator and inhale deeply.
5. If air leaks around the edges, adjust the nosepiece and straps for a better fit

IMPORTANT

Before use, the wearer must read and understand these instructions. Keep for reference.

Single use respirators against solid and liquid aerosols are classified according to one of three categories FFP 1, FFP 2 and FFP 3. Before reading the following information, please check the category to which the respirator belongs, this is indicated on the packaging and on the respirator.

Applications:

FFP1: protection against non-toxic solid and liquid aerosols e.g. oil mists in concentrations up to 4.5 times MAC/OEL/TLV or 4 x APF.

FFP2: Protection against nontoxic and low to average toxicity solid and liquid aerosols eg oil mists in concentrations up to 12 x MAC/OEL/TLV or 10 x APF.

FFP3: Protection against nontoxic and low to average toxicity solid and liquid aerosols eg oil mists in concentrations up to 50 x MAC/OEL/TLV or 20 x APF.

Warning: 1 failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of the respirator and result in illness or death. 2 a properly selected respirator is essential to protect your health. Before using this respirator, consult an industrial hygienist or Occupational Safety professional to determine the suitability for your intended use. 3. this product does not supply oxygen. Use only inadequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%. 4. Do not use when concentrations of contaminants are immediately dangerous to health or life. 5. Do not use in explosive atmospheres. 6. Leave work area immediately and return to fresh air if breathing becomes difficult or dizziness or other distress occurs. 7. Facial hairs or beards and certain facial shapes may reduce the effectiveness of this respirator. 8. Never alter or modify this respirator in anyway. 9. No maintenance necessary. Discard the use respirator after a single use. 10. Store respirators in the display box away from direct sunlight until use.



OCEAN
FOOTPRINT

50/Box

FFP2 NR MASK



CE 0158 EN 149:2001+A1:2009



产品合格证

品 牌: TEYIN 牌
产品名称: 无纺布民用口罩
产品用途: 个人防护 (非医用)
产品型号: TY1729 FFP2 NR
主要材质: 30%熔喷布+70%无纺布
包装规格: 50只/盒
生产批号: 202005001
生产日期: 05/2020
有效期至: 05/2025
执行标准(欧盟): EN149:2001+A1:2009
质检员: 01
生产商: 苏州市特茵不织布有限责任公司
地 址: 江苏省苏州市吴中区木渎镇穹灵路2163号



OCEAN
FOOTPRINT

FFP2 N
MASI



CE 0158 EN 149:2001+A1:2009

TE YI

103号